

### PASS



**PASS: GREEN IS GOOD!** Different sterilizers produce varied results depending on quality of steam, as long as the emulator has turned **GREEN** the parameters have been met.

### UNEXPOSED



### FAIL




**FAIL:** Any **PURPLE** or **GREY** on the indicator dot is a FAIL. If in doubt do not use.

### WET STEAM



**WET:** Emulator laminate will “bubble” when excessive moisture has been present in the sterilizing process.

### SUPERHEATED STEAM

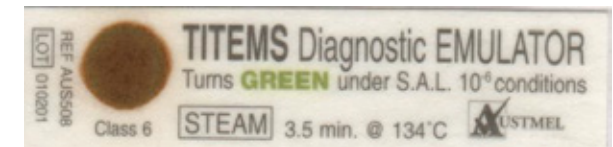

This is a **FAIL**. Emulator will “burn” due to lack of moisture in the steam during sterilization.